


Pinnacle / A70 Series Setup Guide



Instructions on how to setup your Massage Chair.

1. Open the box.
2. The chair will be in the reclined position in the box. Plug the power cord into the chair while it is still in the box.
3. Turn the power switch on the chair to the 'on' position
4. Use the remote control to recline the chair back to the upright position
5. Now you can wheel the chair from the carton to where you would like the massage chair in your home. Example shown below on how to move chair.

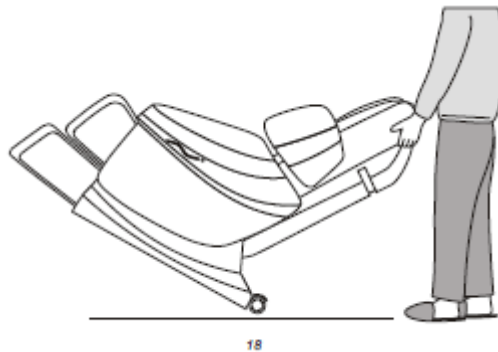

6. Rest the side panel to the side of the chair.

Pinnacle / A70 Series Setup Guide

7. Pull the wires out from the chair and place through hole (as shown below)



8. Now connect the wires coming out from the chair to the wires on the side panel. These will be colour coded or labelled A, B, C etc



9. There will be four white plastic mounting lugs that will go into the frame of the chair. Line these up and put the side panel in place.

Pinnacle / A70 Series Setup Guide

10. Now the side panel is on, you will need to locate the packet of bolts and washers. Use the bolt with the with the allen key heads. The hole locations for the bolts are on the inside of the panel (show below)



11. Insert the front bolt and tighten with the allen key that is provided.



Pinnacle / A70 Series Setup Guide

12. Insert the rear bolt and tighten with allen key provided



13. Now follow the same steps to attach the other side panel.

14. Now locate the small box that contains the shoulder massagers



Pinnacle / A70 Series Setup Guide


15. Remove the plastic material on each of the shoulder massagers by undoing the Philips head screw. When you remove the plastic cover, you will see wires inside.



16 Connect the wires from the massage chair to the wires on the shoulder massagers

17. Fasten the shoulder massagers onto the chair with the bolts provided

18. Now attach the remote control arm to the side of the chair.

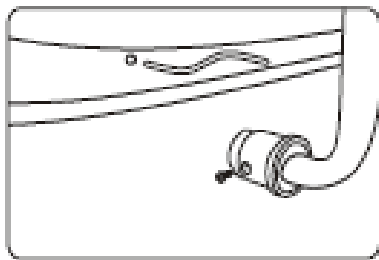

19. Use the screws provided to fix into place

20. Turn the power button on remote control to turn on the chair.

21. Please refer to the user manual on how to operate the massage functions of the chair.