

CONTACT US:

Call us at: 03 9111 0453
backdoctormassagechairs.com.au
info@backdoctormassagechairs.com.au

ULTIMATE CHIRO PRODUCT MANUAL

Thank you for purchasing your Ultimate Chiro massage chair.

You have purchased a chair from Australia's leading massage chair specialist!

This chair was designed to provide excellent relief from back pain, muscle fatigue and all manners of stress.

Before operating your amazing new chair, please read this instruction manual carefully so you can get the most out of your massage experience.

Please pay attention to the important safety warnings and keep this manual for future use.

CONTENTS

Safety and maintenance	01
Name and function of components	03
Functions	04
Method of usage	05
Product specification	16

Safety and maintenance

1. IMPORTANT SAFETY WARNING

- Do not allow children to touch the moveable parts of this product.
- Please only use the grounding power supply included with this product.
- Please unplug the product when not in use, or before cleaning, to avoid injury or causing damage to the product.
- Please operate this product according to this instruction manual.
- Do not use accessories not recommended by the manufacturer.
- This product is not suitable for outdoor use.
- Any other usage that is not listed in this instruction manual is forbidden.
- The recommended usage time of this product is 20 minutes.
- Please do not use the product if the seat cover or leather is broken or damaged.
- Please do not use it if any ventilation holes are covered or blocked.
- Please do not drop anything into this product.
- Always start with the massage function on a lower strength setting before increasing it, in order to avoid injury.

2. Environment for usage

- Keep this product out of high temperature environments. Keep away from fire and long term exposure to the sun.
- Please do not use it in a dusty or caustic environment.
- Please do not use it where there is limited space or poor ventilation.

3. People who are not suitable to use this product

- People who are suffering from osteoporosis are not recommended to use this product.
- People who are suffering from heart disease or carrying electronic medical equipment such as a pacemaker are not recommended to use this product.
- Pregnant women or menstruating women are not recommended to use this product.
- People who have injuries or skin disease are not recommended to use this product.
- Children under the age of 14 or people with mental health issues or disabilities are not recommended to use this machine without supervision.
- People diagnosed with medical conditions are not recommended to use this product.
- People who have come in contact with water or those who are wet are forbidden to use this product.

4. Safety

- Check that the voltage is suitable to the specification of this product.
- Do not pull the plug with wet hands.
- Do not get this product wet or let it come into contact with water. This will avoid electric shock and damage to the product.
- Do not use wet cloth or any wet material to clean the electric parts of this product such as the switch and plug.
- Stay away from this product during power interruptions to avoid any injury if the power is suddenly restored.
- If this product stops working correctly, please stop using it immediately. Contact the retailer.
- If you feel unwell, stop using the product and consult a doctor immediately.

Safety and maintenance

- This product is not intended for use by individuals, including children with reduced physical, sensory and mental capabilities. This product is also not intended for individuals who lack the knowledge or experience to use it, unless there is proper supervision or they have been thoroughly instructed on how to use it.
- Children should be supervised around and while using the product.
- If the supply cord is damaged, it should be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid hazards.


5. Maintenance

- Product maintenance should only be done by approved service agents. Refrain from disassembling the product yourself.
- Always unplug the power cord after product use.
- Do not use this product if the socket is loose.
- If this product is not often used, fold the wires properly and store in a cool, dry and dustless place.
- Keep this product out of high temperature environments. Keep away from fire and long term exposure to the sun.
- If the detachable cord/supply cord is damaged, only use replacements from the manufacturer, its service agent or similarly qualified individuals to avoid hazards.
- Use dry cloth to clean this product. Do not use flammable liquids such as thinner, benzene, or alcohol to clean this product.
- The mechanical components of this product are specially designed and made, it is not necessary to have it regularly serviced.
- To move this product on an uneven surface, it should be lifted. Do not pull or roll it.
- Please use this product intermittently; do not use it continuously for long periods of time.

6. Solutions to ordinary malfunctions

- It is normal to hear the sound of the motor when the massage chair is in use.
- If the controller does not work normally, please check if the plug and socket are connected properly and make sure that the switch is turned on.
- If the set massage time is over, this product will automatically shut down. If this product is continuously used for a long period of time, the temperature protector will automatically shut down the power. When this happens, this product should be used only after 30 minutes of rest.

Name and Function of Components

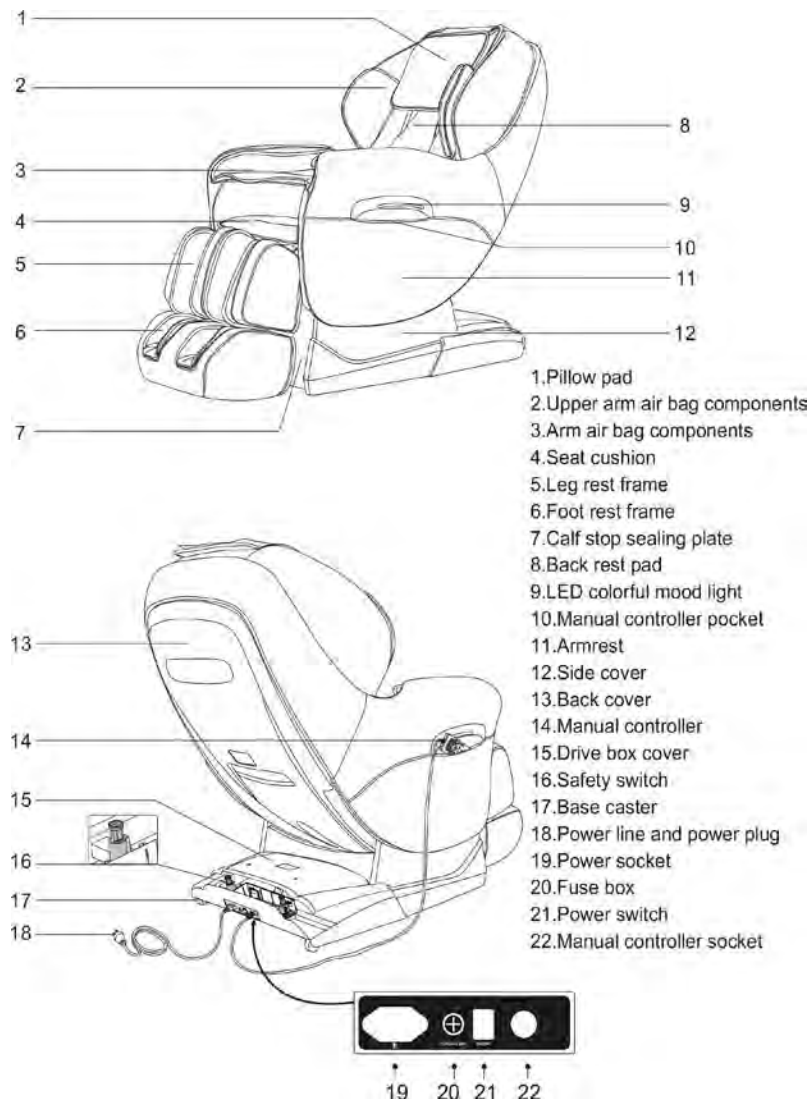

Functions

- The massage mechanism is a set of vertically movable, four-wheel driven, muted, intelligent massage hands.
- Designed with emulation massage methods: shiatsu, kneading, flapping, knocking, simultaneous kneading and flapping.
- Shoulder automatic detection and location setting.
- Set with six kinds of characterised auto massage functions: ache-improving, comfort, ease, fast improving experience massage, upper body, and lower body.
- Memory setting on function M1 and M2.
- Manual massage for the upper body with three options: fixed, partial, and overall massage. Five levels of speed adjustments in the state of flapping, shiatsu and knocking. The width between the two kneading balls is adjustable in three levels: wide, medium and narrow. On partial and fixed points, the massage hands can move up and down to adjust the massage point accurately.
- Air pressure massage function has three modes: full body air pressure, arms (upper arm set with 4 airbags, arm set with 8 airbags), and leg set with 16 airbags with three intensity options.
- Leg rest is designed with roller massage function at the bottom of the foot, with two pairs of papillae on toe root rotating massage; on the arch and heel mastoid skin scraping massage.
- Back heating function: carbon fiber infrared heat source to maintain temperatures of 40 °C to 50 °C.
- Armrest side configuration blue LED colorful artistic lamp.
- Calf rest can extend to fit different body heights.
- Backrest frame uses down sliding technology to deviate from the wall by keeping zero space to save the maximum space possible.
- Leg lifting adjustment, backrest frame lifting adjustment and automatic lay down mode adjustment functions.
- L-shape back track fits the curves of the body.
- Zero gravity function.


Method of Usage



NOTE: This display refers to the actual remote control

Method of Usage


1. Switch power supply on

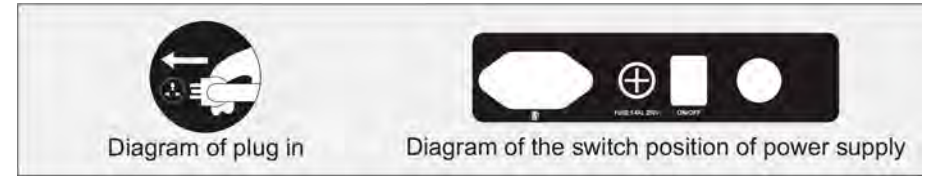

2. Start to massage

- 1) Press power button, massage chair automatically reclines and starts massage.
- 2) Automatic shoulder position and massage acupressure point detection, wait for the testing to be completed.
- 3) If there is a difference with the massage hands and the actual shoulder position, you can adjust the up and down button to a suitable position. Shoulder position has 10 grade adjustments. It will run on automatic massage if there is no adjustment on the shoulder position within 10 seconds.

Button	Description	Display
Adjust shoulder	Move the massage hands upward to a suitable position	
	Move the massage hands downward to a suitable position	

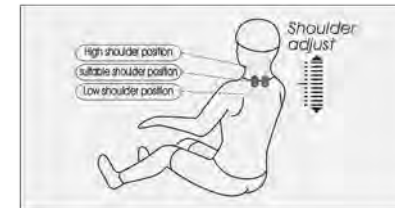

4) Choose massage function to massage

Memory function massage

Press the button "Upper body / M1" or "Lower body / M2" for 3 seconds, then enter into memory massage function "M1" or "M2" to revert to the previous memory massage mode.

Function	Button	Display
Memory Massage	Upper body / M1	Memory
	Lower body / M2	Memory

Method of Usage

- Characterised auto massage

Function	Button	Feature	Display
Characterised Auto Massage		Deep massage on acupressure points to effectively alleviate pain	Auto F1
		Loosens body physique	Auto F2
		Promotes blood circulation	Auto F3
		Quickly experience all massage effects (press 2S to switch to speed press)	Auto F0
		Massage focuses on upper body	Auto F4
		Massage focuses on lower body	Auto F5

- Upper body manual massage and lower body manual massage

1) Upper body manual massage

Function	Button	Adjust Massage Ball	Display
Technique		Speed and Width	
		Speed	
		Speed and Width	
		Speed	
		Speed and Width	
Speed		Speed - Three Adjustable Grades	
Width		Widths - Three Adjustable Grades	

Method of Usage

Function	Button	Feature	Display
Back Stretch		Fixed position (fixed point) massage back and forth	H0
		Small area (partial area) massage back and forth	H1
		Overall massage back and forth	H2
		Upper body massage back and forth	H3
		Lower body massage back and forth	H4
		Massage point adjust (in fixed point and in partial modes so you can adjust the massage position)	Massage hand moves up
			Massage hand moves down

- Air pressure function massage






Function	Button	Description	Display
Air pressure mode		Three air pressure modes (full body, arm, and leg)	Arm Feet
Intensity		Three grade adjustments	Arm Feet

- Other manual functions

Function	Button	Description	Display
Roller		Sole Roller: Sole roller switch, with 3 speed levels available	
Heating		Infrared back heating, feels warm after approximately 3 minutes. Temperatures range from 40 °C to 50 °C.	


Method of Usage

- Other manual functions

Function	Button	Description
Calf rise and drop		Calf Rest Lifting Button: Press this button, calf rest lifts slowly and stops when released.
		Calf Rest Decline Button: Press this button, calf rest declines slowly and stops when released.
Backrest rise and drop		Backrest Lifting Button: Press this button, backrest lifts slowly and stops when released.
		Backrest Decline Button: Press this button, backrest declines slowly, stops when released.
Auto lay down mode		Total of 3 automatic reclined positions. In startup mode, the chair will automatically recline to a comfortable seated position. Pressing the button once will further recline the chair. To achieve the zero gravity position, press the button again.

3. Turn off power, stop massage

- 1) Pressing the power button during the massage will stop all functions, and the backrest and footrest will return to its original position automatically. Meanwhile, you can set up the memory function. If the rated working time finishes, the massage functions will turn off but the backrest and footrest won't return to their original positions.
- 2) Memory Store Function: After the rated working time or product is switched off, the hand controller displays "Memory", asking the user whether they would like to store the massage mode before turning off the machine. You can press the memory function buttons "M1" or "M2" to store the massage mode.
- 3) Cut off the power of the whole machine.

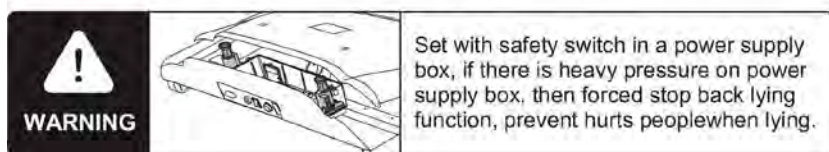

Method of Usage

4. Armrest disassembly method

Note: This method is only used if you cannot easily move due to small space. Please do not arbitrarily disassemble.

- Open the master box, take down the calf rest. (Figure 1)
- Remove the back cushion. Ensure the heat wire in the cushion is unplugged before removing. (Figure 2)

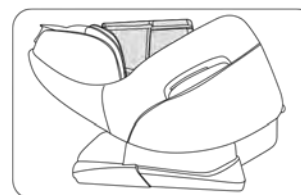


Figure 1

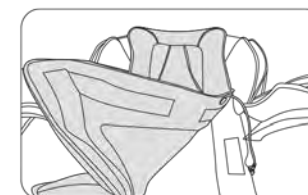


Figure 2

- Zip the backrest kneading cloth open completely, and put the cloth on the top of the chair (Note: Please remember to prevent the kneading cloth from getting dirty or damaged). (Figure 3)
- The calf rest decoration cover is a card slot design; please make an effort to pull it when disassembling. (Figure 4)

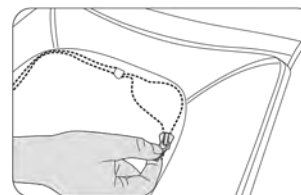


Figure 3

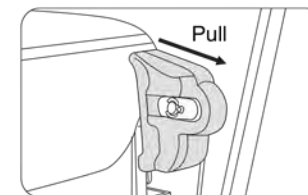


Figure 4

- Remove the 8 screws from the front dam-board, NOT the 4 screws on the bottom. (Figure 5)
- Open the front dam-board, unplug the joint of the air pipe and the power cord that corresponds to the armrest. (Figure 6)

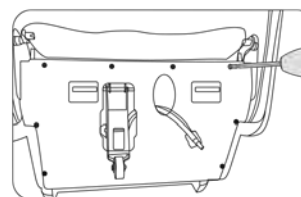


Figure 5

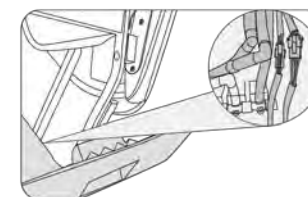


Figure 6

Method of Usage

- Remove the screw on the front-end of the armrest (Note: The screw is in the side of steel frame). (Figure 7)
- Remove the screw on the front-end of the armrest (Note: Be careful not to miss the fixed plastic block). (Figure 8)



Figure 7

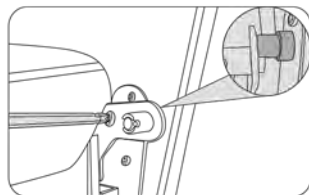


Figure 8

- Remove the screw on the back-end of the armrest (Note: Zip the backrest kneading cloth open completely to find it). (Figure 9)
- Gently lift up the front-end of the armrest and pull out the buckle. (Figure 10)

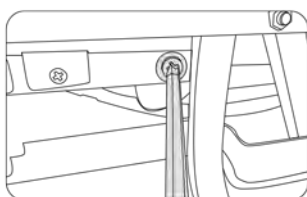


Figure 9

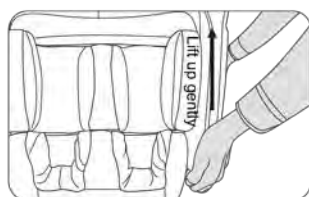


Figure 10

- Gently pull forward the back-end of the armrest to pull out the decoration cover, then lift up the back-end of the armrest to pull out the buckle. (Figure 11)
- Unplug the joint of the upper armrest air pipe to remove the armrest completely. (Figure 12)

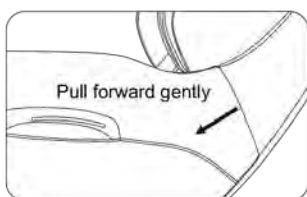


Figure 11

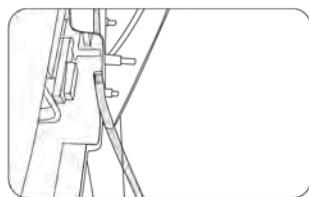


Figure 12

Method of Usage

5. Armrest installation method

- Connect the air pipe on the back-end of the armrest to the upper arm. (Figure 1)
- Put the air pipe and power cord into the steel frame (Note: Avoid getting the air pipe stuck). (Figure 2)

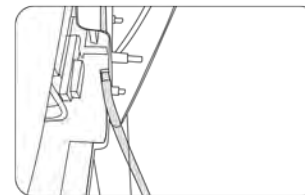


Figure 1



Figure 2

- Hook the back-end of the armrest on the steel frame (Note: There are two; place it on the upper one). (Figure 3)
- Insert the back-end of the armrest into the decoration cover of the upper arm. (Figure 4)

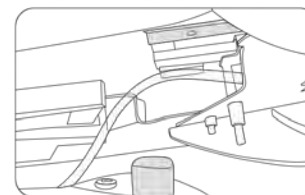


Figure 3

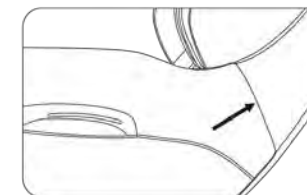


Figure 4

- Hook the front-end of the armrest on the steel frame and lock the buckle. (Figure 5)
- Gently push the armrest down, and double check that the hook is well locked. (Figure 6)

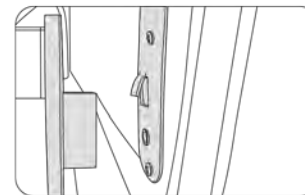


Figure 5

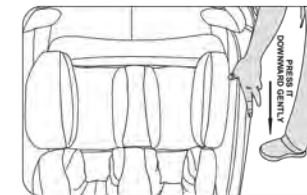


Figure 6

- Put the plastic block between the armrest and the calf rest pothook, then tighten the up and down screws. (Figure 7)
- Tighten the screws on the back-end of the armrest. (Figure 8)

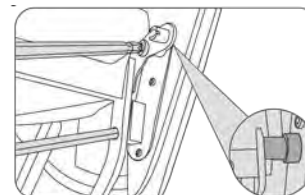


Figure 7

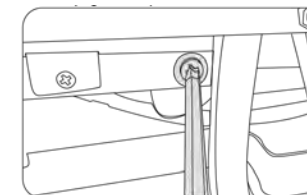


Figure 8

Method of Usage

- Connect the joint of the air pipe and power cord (Note: There are marks on the air pipe and the cable connectors are different sizes). (Figure 9)
- Pull the calf rest air pipe and power cord through the hole of the front dam-board, then tighten the screws of the dam-board. (Figure 10)

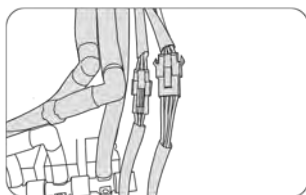


Figure 9

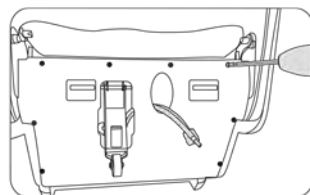


Figure 10

- Zip up the backrest kneading cloth. (Figure 11)
- Plug the heat plug of the back cushion into the heat hole on the backrest, and put the back cushion tidily on the backrest. (Figure 12)

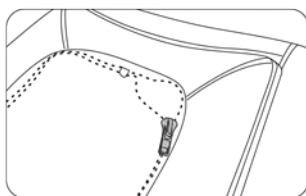


Figure 11

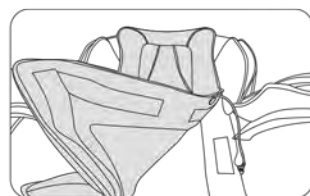


Figure 12

- The calf rest decoration cover is a card slot design, make an effort to push it during assembly (Note: Keep the cover tight with the side), armrest installation is completed. (Figure 13)

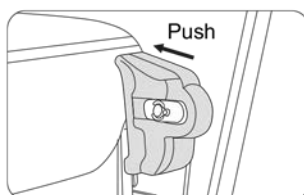


Figure 13


Method of Usage

6. Calf installation

- Install the rotating shaft in-calf with the fixed axis first. Note: The hole should be aligned with an R shape pin assembly. (Chart 1)
- After calf is connected and aligned with the fixed axis, insert it. (Chart 2)




(chart 1)

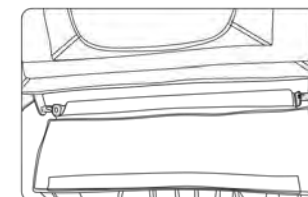


(chart 2)

- Plug in the R pin, then move to the other side of the calf, and plug in the next R pin. (Chart 3)
- Finished. (Chart 4)

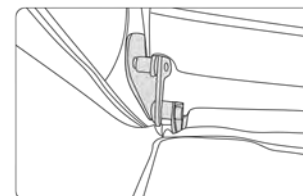


(chart3)

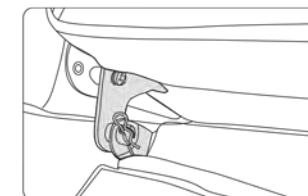


(chart4)

NOTE:

- While the machine is reclining, do not apply your body weight to the calf rest. (Chart 1)
- When going to sit down, ensure that the massage hands are not in the seat area to avoid discomfort. (Chart 2)

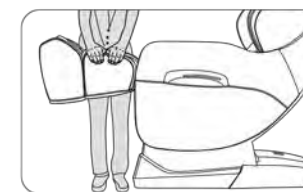

(chart1)

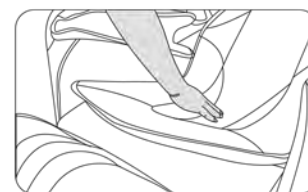


(chart2)

Method of Usage


7. How to use the pillow and backrest cushion

Using pillows can reduce the kneading massage intensity on the shoulder and neck. You decide whether the pillow pad is needed according to your preference (recommended). The backrest cushion and backrest are connected by zipper (1), head cushion and backrest pad are connected by thread gluing (2).

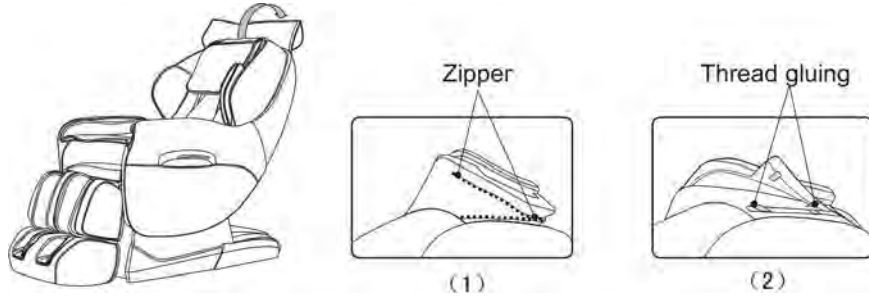

8. Floor protection

Putting the heavy massage chair on a wooden floor for an extended period may damage the floor, so please place a carpet or other flooring material to avoid this damage.

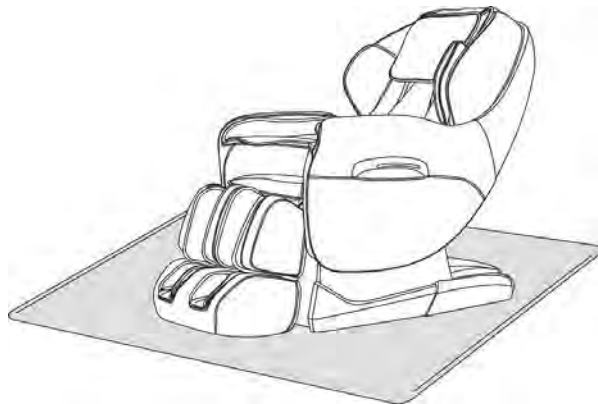


Caution: Do not move the chairs with its wheels on the wooden floor, on rough ground or in a narrow space. Two people are required to lift the chair. Please do not lift armrest directly.

Method of Usage

9. Method of movement

Make sure that all wires are not on the ground. Tilt the backrest backward to a certain degree (with the gravity centre resting on the wheels), push the chair forward or backward with your hands and finally place the chair back in its normal position in a slow and gentle manner.

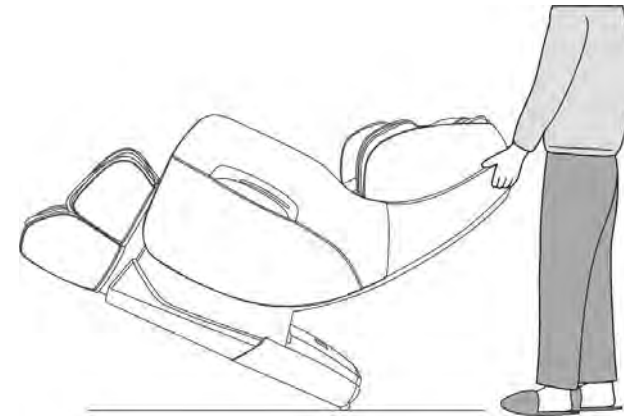

Note: Be sure to turn off the power switch first and then unplug the power line and hand controller connecting line before moving the chair.

Product Specification

- Model: Ultimate Chiro
- Rated voltage: 220 – 240V ~ 50 Hz/60 Hz
- Rated power input: 150W
- Rated time: 20 Min
- Safety structure: Class I

Material introduction: PVC, PA, steel parts and electric and electronic parts

MATERIAL	PU	LEATHER	CLOTH	WOOD